

Saunders Lumbar HomeTrac®



10199

Lumbar Traction

The HomeTrac name is synonymous with quality home traction.

We've responded to the need for a home lumbar traction device that's even more compact and affordable.

- Variety of patient positioning options
- Replicates clinical traction
- Allows traction in supine position
- Provides therapeutic force up to 200 pounds
- Patented pump allows immediate pressure relief
- Ensures continuity between home and clinical treatments

Saunders Lumbar HomeTrac®

Lumbar Traction

Product Profile

Features and Benefits

- Effectively applies up to 200 pounds of traction
- Friction-free surface ensures smooth motion
- Folds to only 23" x 24" x 3" for easy storage in a sturdy carry case
- Weighs only 16 pounds
- Comes with instructional video and user's guide, written with the patient in mind



- All Saunders home traction products feature our patented pump that allows immediate pressure relief, if necessary, during treatment



Quality home lumbar traction is as easy as . . . 1 - 2 - 3

- Open the unit – it self-locks into position
- Secure the belts – they're already attached
- Pump it up – effective traction, smooth and quick

| Code | Description |
|--------------|-------------------------------|
| LMBR TRAC DX | Deluxe Lumbar Traction Device |
| 10911 | Accessory Kit |

Indications

- Pain in back or neck
- Herniated or protruding discs
- Degenerative disc disease
- Posterior facet syndrome
- Sciatica
- Joint hypomobility
- Sub acute joint inflammation
- Nerve root impingement
- Paraspinal muscle spasm

Contraindications

- Acute or traumatic injury
- Spinal instability
- Fractures
- Extruded disc fragmentation
- Severe rheumatoid arthritis
- Spinal cord compression
- Infections or inflammatory disease
- Serious pathology, disease or malignancy
- Spreading and/or aggravation of patient's symptoms

CAUTION

Federal law restricts this device to sale by, or on the order of, a practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.