

Saunders Cervical HomeTrac®



Cervical Traction

Our sleek new design features self-adjusting neck wedges that eliminate the need for awkward adjustment knobs. The new wedges provide closer, more secure contact for a perfect fit every time. Lie down, fasten the head strap and let the HomeTrac's comfortable wedges self-adjust by rotating on their base. Never has home cervical traction been simpler.

- Ensures optimum angle for traction treatment
- Provides therapeutic force up to 50 pounds
- Replicates clinical traction
- Self-adjusting neck wedges
- Allows traction in supine position
- Ensures continuity between home and clinical treatments

Saunders Cervical HomeTrac®

Cervical Traction

Product Profile

Features and Benefits

- Directs traction forces toward the occiput (back of the head) thereby preventing compression of the TMJ through the chin
- Applies up to 50 pounds of traction
- Ensures safety - user has total control of force at all times
- Enables easy home traction without assistance
- Provides a comfortable and cost-effective option to continuing clinical traction treatments
- Includes instructional video, user's guide and deluxe carrying case
- Patented pump design incorporates an easy-to-use "click-in-place" mechanism for pumping, sustaining and releasing the traction



- Gauge reads in pounds/kg of traction to provide feedback to the user and to promote consistency in treatment



- Self-adjusting neck wedges rotate through a 20° arc and are removable for easy cleaning or replacement



- Adjustable extension foot allows for additional traction angles of 20° and 25°



Before



During

- The cervical spine before traction and during traction applied by the Saunders Cervical HomeTrac®

Indications

- Pain in back or neck
- Herniated or protruding discs
- Degenerative disc disease
- Posterior facet syndrome
- Sciatica
- Joint hypomobility
- Sub acute joint inflammation
- Nerve root impingement
- Paraspinal muscle spasm

Contraindications

- Acute or traumatic injury
- Spinal instability
- Fractures
- Extruded disc fragmentation
- Severe rheumatoid arthritis
- Spinal cord compression
- Infections or inflammatory disease
- Serious pathology, disease or malignancy
- Spreading and/or aggravation of patient's symptoms

Code	Description
CTRAC DLX	Saunders Cervical HomeTrac
100625-R2	Replacement neck wedges

Precautions

It is recommended that the force be increased gradually over several sessions. Slight, post-treatment muscle soreness in the neck is common; however excessive soreness or an increase in peripheral symptoms may result if the force is increased too quickly.

CAUTION

Federal law restricts this device to sale by, or on the order of, a practitioner licensed by the law of the State in which he/she practices to use or order the use of the device.